



# DuraBase

## DENTURE RELINE MATERIAL

High color stability

Fast setting

Cross-linked for improved craze resistance



Reliance Dental Manufacturing, LLC.

For Product Updates & Information, sign up at [reliancedental.net/contact/updates](https://reliancedental.net/contact/updates)

## CHAIRSIDE INSTRUCTIONS

1. Clean and roughen denture area to be relined. Dry denture completely.
2. Lubricate the labial and buccal surfaces within 1/8" of the peripheral border.
3. Paint reline area with DuraBase Repair Liquid.  
This assures a chemical bond.
4. Recommended powder/liquid proportions:  
Powder            10 ml        - shorter plastic vial  
Liquid            5 ml            - taller plastic vial  
  
Add liquid to powder in mixing cup and stir thoroughly for 30 seconds.
5. Wait approximately 1 to 3 minutes and apply mixture to denture.
6. Insert denture in mouth and have patient close mouth lightly into occlusion.
7. After approximately 3–4 minutes in the mouth, muscle-trim the borders, then remove the denture and trim the excess. Rinse the denture in cold tap water and reseat in the patient's mouth. Remove and let cure on bench.
8. DuraBase sets in approximately 15 minutes after curing. Trim any excess material and polish denture.
9. To reduce residual taste, place denture in hot water for 10 minutes at 130° F / 55° C or in a pressure cooker at 130° F / 55° C for 10 minutes at 15 psi.

**PACKAGE CONTAINS:** 4 oz. Powder; 4 oz. Liquid; Lubricant; Dropper



Reliance Dental Manufacturing, LLC • 5805 W. 117th Place • Alsip, IL 60803

European Representative: Richard L. Milano Via Tiepolo 10126 Torino, Italy