



DuraGlaze

GLAZE & THINNER

Use on composite or acrylic

Cures in or out of the mouth

Provides polished surface

Seals porous surfaces

Sets & hardens in 20 seconds



Reliance Dental Manufacturing, LLC.

For Product Updates & Information, sign up at reliancedental.net/contact/updates

TECHNIQUE

1. Finish and polish restoration.
2. Make sure surface is clean and dry before applying glaze.
3. Dip brush into bottle, wipe excess off brush on bottle rim, or on ceramic or glass slab, and then coat composite or acrylic surface with straight, even strokes. Allow glaze to dry before applying a second coating.
4. Clean brush in thinner (solvent) after each application, to keep brush clean and bristles soft.
5. For additional coats, follow step 3.
6. If removal is necessary, grind off with polishing wheel.
7. To maintain free flowing consistency, glaze liquid should be kept thin with periodic addition of solvent. Use dropper for addition of thinner (solvent).

Keep bottles tightly sealed when not in use.

Keep away from open flame.

GLAZE is a fast setting liquid acrylic resin which bonds to the resin substrate and air dries to a clear hard surface.

10CC PACKAGE CONTAINS: 10 cc Glaze; 10 cc Thinner

1 OZ PACKAGE CONTAINS: 1 oz Glaze; 1 oz Thinner



Reliance Dental Manufacturing, LLC • 5805 W. 117th Place • Alsip, IL 60803

European Representative: Richard L. Milano Via Tiepolo 10126 Torino, Italy