



# DuraSeal

## TEMPORARY FILLING ACRYLIC

Self-curing acrylic

Applies easily using the brush technique  
for quick application

Remains semi-pliable for easy removal

Natural tooth color



Reliance Dental Manufacturing, LLC.

For Product Updates & Information, sign up at [reliancedental.net/contact/updates](https://reliancedental.net/contact/updates)

# INSTRUCTIONS

## SIMPLE, POSITIVE TECHNIQUE

### BRUSH METHOD

Place a small quantity of powder in one dappen dish and liquid in another. Alternately dip brush in liquid, then in the powder. Deposit this bead of mix on cavity floor. Repeat the operation until properly filled. Let set, check bite and trim.

### MIX METHOD

Place powder in dappen dish and add sufficient liquid to make a thin mix. Stir a few seconds. Apply to cavity with brush or instrument. (Mix on glass slab, if preferred.) When DuraSeal has set, check bite and trim excess.

If phenol, eugenol or thymol is used as a medicament in a preparation, place a pellet of cotton in the cavity before capping with DuraSeal.

**DURASEAL** sets in approximately 4 minutes with virtually no heat and seals completely. It is quickly removed by heating the tip of an instrument, inserting the tip into the DuraSeal and lifting the DuraSeal out in one piece, even with an undercut. Use only DuraSeal liquid with DuraSeal powder.

**STANDARD PACKAGE:** 2 oz. Powder; 2 oz. Liquid; Dropper

**ECONOMY PACKAGE:** 8 oz. Powder; 8 oz. Liquid; Dropper



Reliance Dental Manufacturing, LLC • 5805 W. 117th Place • Alsip, IL 60803

European Representative: Richard L. Milano Via Tiepolo 10126 Torino, Italy