



# DuraTray II

## CUSTOM TRAY RESIN

Non-Sticky  
Highly Adaptable and Workable  
Reduces Fabrication Time  
Cures in 6 Minutes



Reliance Dental Manufacturing, LLC.

For Product Updates & Information, sign up at [reliancedental.net/contact/updates](https://reliancedental.net/contact/updates)

# Innovative Custom Tray Resin that can cut working time in half

This system is designed to produce a non-sticky, incredibly adaptable dough in about 30 seconds. It minimizes fabrication work, is workable right up to the cure point, and is constantly self-adhesive. A handle can be added at any point without having to add monomer or mix additional resin.

## INSTRUCTIONS

1. Pour liquid into orange vial. Fill to top of vial and pour into mixing cup.
2. Pour powder into blue spoon, slightly under-filling. The powder should be just below the top of the spoon. Pour into cup.
3. Mix thoroughly for 30 seconds.
4. DuraTrayII is now ready to adapt.

**1 LB PACKAGE CONTAINS:** 16oz. Powder; 4 oz. Liquid; Mixing Cup; Spoon; Dropper



Reliance Dental Manufacturing, LLC • 5805 W. 117th Place • Alsip, IL 60803

European Representative: Richard L. Milano Via Tiepolo 10126 Torino, Italy