

DuraLay^{CE}

Temporary Crown & Bridge Kit

Shades: 62, - 65, - 69, - 77;
2 - 2 oz. LIQUIDS
for
Temporary Bridges,
Partials,
Jacket Crown, Etc.

DuraLay Crown & Bridge resin, sets up rapidly in only 5 minutes to a strong hard acrylic which is easy to trim. As most doctors know from using the red DuraLay Inlay Pattern Resin, there is minimal shrinkage which makes this DuraLay ideal for temporary crowns and bridges.

Use only DuraLay Liquid
with DuraLay Powder



*Since 1894: Over 100 years
of Reliable Service to the Dental Community*

INSTRUCTIONS:

DuraLay is a cold-curing resin supplied as powder and liquid. The powder is methyl methacrylate polymer. The liquid is methyl methacrylate monomer.

The mixing proportion is 3 parts powder to 1 part liquid by volume; 20 gr. powder to 10 cc. liquid by weight.

1. Take an impression prior to doing any preparatory work, using alginate, rubber base, silicone or hydrocolloid. Lay aside and keep moist.
2. Complete all tooth preparations and then lubricate teeth with vaseline or a silicone lubricant.
3. Dry impression thoroughly. Apply sufficient liquid directly to impression with medicine dropper and add powder. Dispense powder from plastic bottle by shaking lightly until liquid is absorbed. More liquid and powder can be added if needed. For temporary crown, using crown form, fill form half full of liquid and add powder until liquid is absorbed.
4. After impression or crown form is filled with DuraLay, wait one minute for dull appearance. Reset in mouth in proper position. Working time is 3 minutes from start to mix. The impression can then be removed from the mouth without distorting. Set impression on bench for complete hardening, which is approximately 5 minutes from the start of mix.
5. Remove DuraLay from impression or crown form and trim excess. The temporary crown or bridge is finished and polished with stones or carbine burs and conventional composite polishing instruments, then cemented into place.

Precautions:

Surrounding gingival tissues should be protected from the DuraLay resin. Exposed vital dentin should also be protected from DuraLay resin. May cause skin irritation. Use with adequate ventilation.

Individual replacements may be bought from your dealer.



Dental Mfg. Co.

P.O. Box 38
Worth, IL 60482

(708) 597-6694 FAX (708) 597-7560

RICHARD L. MILANO VIA TIEPOLO, 10126 TORINO, ITALY