

Dura Ortho - Acrylic

**Orthodontic Resin for
the Construction of
Orthodontic Retainers
and Appliances**

*Chemically related to
DURALAY for these
unique qualities*

- Minimum Shrinkage
- Dense, Strong Acrylic
- Fast Setting
- Superior Clarity

Available in Pink or Clear


Reliance
Dental Mfg. Co.
Worth, IL 60482

INSTRUCTIONS FOR SALT/PEPPER TECHNIQUE

1. Bend wire in conventional manner.
2. Soak model in water until no bubbles appear. Apply separating liquid to stone cast.
3. Apply wire on cast and hold in place with sticky wax.
4. Moisten cast surface with liquid, then spray on powder. Alternate liquid and powder for desired shape and thickness. Finish with liquid for best result.
5. For best results, use of a pressure polymerizer or a hydro flask is recommended.



Dental Mfg. Co.

P.O. Box 38
Worth, IL 60482

(708) 597-6694 FAX (708) 597-7560