

# **DuraBase**

---

---

## ***Denture Reline Material***

- high color stability
- fast setting
- cross-linked for improved craze resistance



*Since 1894: Over 100 years  
of Reliable Service to the Dental Community*

## INSTRUCTIONS:

1. Clean and roughen denture area to be relined. Dry denture completely.
2. Lubricate the labial and buccal surfaces within 1/8" of the peripheral border.
3. Paint reline area with DuraBase Repair Liquid. This assures a chemical bond.
4. Recommended powder/liquid proportions:  
(by volume)  
Powder      10 ml    - shorter plastic vial  
Liquid        5 ml    - taller plastic vial  
Add liquid to powder in mixing cup and stir thoroughly for 30 seconds.
5. Wait approximately 1 to 3 minutes and apply mixture to denture.
6. Insert denture in mouth and have patient close mouth lightly into occlusion.
7. After approximately 3-4 minutes in the mouth, muscle-trim the borders, then remove the denture and trim the excess. Rinse the denture in cold tap water and reseat in the patient's mouth. Remove and let cure on bench.
8. DuraBase sets in approximately 15 minutes after curing. Trim any excess material and polish denture.
9. To reduce residual monomer taste, place denture in hot water for 10 minutes at 130° F or in a pressure cooker at 130° F for 10 minutes at 15 psi.



*Dental Mfg. Co.*

P.O. Box 38  
Worth, IL 60482

(708) 597-6694    Fax (708) 597-7560