



## GLAZE & THINNER

GLAZE can be brushed onto any clean composite or acrylic material, **in or out of the mouth**. Gives a polished surface. Seals porous surface of composites and acrylics. Sets and hardens in about 20 seconds.

### TECHNIQUE:

1. Finish and polish restoration.
2. Make sure surface is clean and dry, before applying glaze.
3. Dip brush into bottle, wipe excess off brush on bottle rim, or on ceramic or glass slab, and then coat composite or acrylic surface with straight, even strokes. Allow glaze to dry before applying second coating.
4. Clean brush in thinner (solvent) after each application, to clean brush and keep bristles soft.
5. For additional coats, follow step 3.
6. If removal is necessary, grind off with polishing wheel.
7. To maintain free flowing consistency, glaze liquid should be kept thin with periodic addition of solvent. Use dropper for addition of thinner (solvent).

Keep bottles tightly sealed when not in use.

Keep away from open flame.

GLAZE is a fast setting liquid acrylic resin which bonds to the resin substrate and air dries to a clear hard surface.

**RELIANCE DENTAL MFG. COMPANY**  
Worth, Illinois