



DuraSeal

Temporary Filling Acrylic



DuraSeal offers:

- Self-curing acrylic
- Applies easily using the brush technique for quick application.
- Remains semi-pliable for easy removal.

- **DuraSeal** is ideal for temporary inlays, onlays, and fillings of all types.
- **DuraSeal** can also be used for temporary crowns and bridges.
- **DuraSeal** sets in approximately 4 minutes with virtually no heat and seals completely. It is quickly removed by heating the tip of an instrument, inserting the tip into the **DuraSeal** and lifting the **DuraSeal** out in one piece, even with an undercut.

INSTRUCTIONS

SIMPLE, POSITIVE TECHNIQUE



BRUSH METHOD—Place a small quantity of powder in one dappen dish and liquid in another. Alternately dip brush in liquid, then in the powder. Deposit this bead of mix on cavity floor. Repeat the operation until properly filled. Let set, check bite and trim.



MIX METHOD—Place powder in dappen dish and add sufficient liquid to make a thin mix. Stir a few seconds. Apply to cavity with brush or instrument. (Mix on glass slab, if preferred.) When DuraSeal has set, check bite and trim excess.



If phenol, eugenol or thymol is used as a medicament in a preparation, place a pellet of cotton in the cavity before capping with DuraSeal.

CE

*Use only DuraSeal liquid
with DuraSeal powder.*

RELIANCE DENTAL MFG. CO.