

---

# ***DuraLay II***

## **QUICK CURE LAB PATTERN RESIN**

- Sets in only 3 minutes
- Less shrinkage than other resins
- A new dimension in ease of handling
- No distortion
- Burns out clean
- Very fine powder



**DENTAL MFG. CO.**

Worth, IL 60482

## **DuraLay II** BRUSH METHOD

- Pour a portion of DuraLay II powder into one dappen dish and about 6 drops of DuraLay II liquid into another.
- Dip small brush in liquid and then into center of powder to pick up a bead on tip of brush.
- Deposit DuraLay II on preparation and build up by repeating the procedure.

## USES OF **DuraLay II**

- Tapered and telescope crowns.
- Inlay, palatal and lingual bars, connectors and clasp.
- Adhesion bridge.
- Connecting joints on wax up stage for bridge work.
- Aligning joints to be soldered.

**PACKAGE CONTAINS: 4 oz. Powder, 4 oz. Liquid  
Lubricant & Dropper**



*Dental Mfg. Co.*

P.O. Box 38

Worth, IL 60482

(708) 597-6694

FAX (708) 597-7560