

CE

Dura-Liner II

Denture Reline Material

- Low Exothermic Reaction
- Improved Color Stability
- Fast Setting
- Cross-Linked for Improved Craze Resistance



*Since 1894: Over 100 years
of Reliable Service to the Dental Community*

INSTRUCTIONS:

1. Clean and roughen denture area to be relined. Dry denture completely.
2. Lubricate the labial and buccal surfaces within 1/8" of the peripheral border.
3. Paint reline area with DuraLiner Bonding Liquid. This assures a chemical bond.
4. Recommended powder/liquid proportions:
by volume
Powder 10ml -shorter plastic vial
Liquid 7ml -taller plastic vial
Add liquid to powder in mixing cup and stir thoroughly for 30 seconds.
5. Wait approximately 1 minute and apply mixture to denture.
6. Insert denture in mouth and have patient close mouth lightly into occlusion.
7. After approximately 3-4 minutes in the mouth, muscle-trim the borders, then remove the denture and trim the excess. Rinse the denture in cold tap water and reseat in patient's mouth.
8. DuraLiner II sets in approximately 12 minutes when cured in the mouth. Remove denture after curing, trim any excess material and polish denture.
9. To reduce residual monomer taste place denture in hot water for 10 minutes at 130° F or in a pressure cooker at 130° F for 10 minutes at 15 psi.

RICHARD L. MILANO VIA TIEPOLO, 10126 TORINO, ITALY



Dental Mfg. Co.

P.O. Box 38
Worth, IL 60482

(708) 597-6694 FAX (708) 597-7560