


DuraLay

PLASTIC PINS FOR POST & CORE

PKG. 50 PIECES

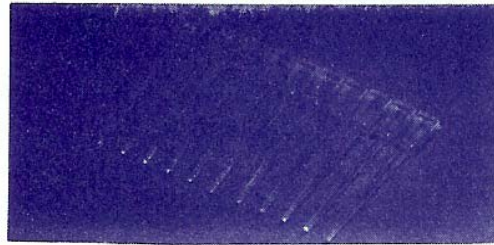

- Serrated so *DuraLay* will readily adhere
- Tapered for easy insertion into the canal
- Complete burnout with *DuraLay* Resin, leaving no residue



Dental Mfg. Co.

5805 W. 117th Place
Worth, IL 60482

1. Prepare canal to desired length and size.
2. Lubricate the canal with DuraLay Lubricant or mineral oil.
3. Make a mix of DuraLay and apply it into the canal or use the brush technique and paint DuraLay into the canal.
4. Dip the DuraLay pin into DuraLay Liquid, then add DuraLay onto the pin using the brush technique.
5. Insert into the canal and remove excess DuraLay. DuraLay will set up completely in about 5 minutes. Before it is set up (approx. 4 minutes) remove pattern and reinsert into the canal. This will facilitate the final removal of the pattern if undercuts are present.
6. Sprue and cast.



*Since 1894: Over 100 years
of Reliable Service to the Dental Community*