



DuraLiner II

HARD CHAIRSIDE
DENTURE RELINE

Low Exothermic Reaction

Improved Color Stability

Fast Setting

Cross-Linked for Improved Craze Resistance



Reliance Dental Manufacturing, LLC.

For Product Updates & Information, sign up at reliancedental.net/contact/updates

INSTRUCTIONS

1. Clean and roughen denture area to be relined. Dry denture completely.
2. Lubricate the labial and buccal surfaces within 1/8" of the peripheral border.
3. Paint reline area with DuraLiner Bonding Liquid.
This assures a chemical bond.
4. Recommended powder/liquid proportions:

Powder	10ml	-shorter plastic vial
Liquid	7ml	-taller plastic vial

Add liquid to powder in mixing cup and stir thoroughly for 30 seconds.
5. Wait approximately 1 minute and apply mixture to denture.
6. Insert denture in mouth and have patient close mouth lightly into occlusion.
7. After approximately 3–4 minutes in the mouth, muscle-trim the borders, then remove the denture and trim the excess. Rinse the denture in cold tap water and reseat in patient's mouth.
8. DuraLiner II sets in approximately 12 minutes when cured in the mouth. Remove denture after curing, trim any excess material and polish denture.
9. To reduce residual taste, place denture in hot water for 10 minutes at 130° F / 55° C or in a pressure cooker at 130° F / 55° C for 10 minutes at 15 psi.

PACKAGE CONTAINS: 4 oz. Powder; 4 oz. Liquid



Reliance Dental Manufacturing, LLC • 5805 W. 117th Place • Alsip, IL 60803

European Representative:

Kentzler-Kaschner Dental GMBH

Muhlgraben 36d • D-73479 Ellwangen - Germany

Tel: +49 7961 907305 • kkd.topdent.de