



Dura Conditioner

TISSUE CONDITIONER
FUNCTIONAL IMPRESSION MATERIAL

Treats and Tones Irritated Tissue
Generates a Functional Impression



Reliance Dental Manufacturing, LLC.

For Product Updates & Information, sign up at reliancedental.net/contact/updates

INSTRUCTIONS

1. Clean and check the denture for centric occlusion and vertical coverage. Relieve all undercuts and pressure areas.
2. Fill the shorter plastic vial with powder and taller plastic vial with liquid and mix together for 1 minute. After standing for an additional minute, pour into denture.
3. Rinse the patient's mouth with cold water and insert the denture. Have the patient close gently into occlusion.
4. After another 3–4 minutes, have patient talk or read aloud to generate a functional impression. After 10 minutes, remove denture and remove overflow material.
5. Examine denture for any pressure areas. If denture base material shows through, indicating pressure area, relieve the area with cc trimmer or carbide bur. These areas may be filled by the brush technique or by making another small mix of DuraConditioner.
6. Have patient return periodically to check progress of treatment until the tissue response becomes finalized.
7. For functional impression, have patient wear denture for at least 5 hours up to a maximum of 24 hours. Pour stone model as soon as possible after removal of impression.

PACKAGE CONTAINS: 4 oz. Powder; 4 oz. Liquid



Reliance Dental Manufacturing, LLC • 5805 W. 117th Place • Alsip, IL 60803

European Representative:
Kentzler-Kaschner Dental GMBH
Muhlgraben 36d • D-73479 Ellwangen - Germany
Tel: +49 7961 907305 • kkd.topdent.de