



Dura Ortho-Acrylic

ORTHODONTIC ACRYLIC

Minimum Shrinkage

Dense, Strong Acrylic

Fast Setting

Superior Clarity

Available in Pink or Clear



Reliance Dental Manufacturing, LLC.

For Product Updates & Information, sign up at reliancedental.net/contact/updates

Orthodontic Resin for the Construction of Orthodontic Retainers and Appliances

INSTRUCTIONS FOR SALT/PEPPER TECHNIQUE

1. Bend wire in conventional manner.
2. Soak model in water until no bubbles appear. Apply separating liquid to stone cast.
3. Apply wire on cast and hold in place with sticky wax.
4. Moisten cast surface with liquid, then spray on powder. Alternate liquid and powder for desired shape and thickness. Finish with liquid for best results.
5. For best results, use of a pressure polymerizer or hydro flask is recommended.

STANDARD PACKAGE CONTAINS: 4 oz. Powder; 4 oz. Liquid

LAB PACKAGE CONTAINS: 16 oz. Powder; 8 oz. Liquid



Reliance Dental Manufacturing, LLC • 5805 W. 117th Place • Alsip, IL 60803

European Representative: Richard L. Milano Via Tiepolo 10126 Torino, Italy