



# DuraLay II

QUICK CURE LAB PATTERN RESIN

Sets in only 3 minutes

Less shrinkage than other resins

A new dimension in ease of handling

No distortion

Burns out clean

Very fine powder



Reliance Dental Manufacturing, LLC.

For Product Updates & Information, sign up at [reliancedental.net/contact/updates](https://reliancedental.net/contact/updates)

# The DuraLay II Brush Method

## APPLICATION

1. Pour a small portion of DuraLay II powder into one dappen dish and about 6 drops of DuraLay II liquid into another.
2. Dip small brush in liquid and then into center of powder to pick up a bead on tip of brush.
3. Deposit DuraLay II on preparation and build up by repeating the procedure.

## USES OF DURALAY II

1. Tapered and telescope crowns.
2. Inlay, palatal and lingual bars, connectors and clasp.
3. Adhesion bridge.
4. Connecting joints on wax up stage for bridge work.
5. Aligning joints to be soldered.

**PACKAGE CONTAINS:** 4 oz. Powder; 4 oz. Liquid; Lubricant; Dropper



Reliance Dental Manufacturing, LLC • 5805 W. 117th Place • Alsip, IL 60803

European Representative: Richard L. Milano Via Tiepolo 10126 Torino, Italy